










Dr. Appt 1

-  Pre-Op Scans
- ▼
-  Photos
(Full Face relaxed, Full Face Smiling, Close up relaxed, Close up smiling, retracted centric, retracted slightly opened)
- ▼
-  Communicate Goals of Case
- ▼
-  **Pick LVI Smile Style**
- ▼
-  **Determine Treatment Plan**
(tooth numbers, veneer/crown, zirconia/lisi, layered/monolithic)
- ▼
-  **If bleaching** Do NOW
- ▼
-  **If changing VDO**
Communicate how much
- ▼
-  Order Ideal or additive only wax up, Prep Guides and/or Temp Stents
- ▼
-  Schedule Custom shade consult (Lab can do photos) if desired

Lab Step 1

- Follow instructions from Dr. Step 1 to design diagnostic wax up, prep guides, temporary stents
- ▼
- Preview of wax up sent for approval, one redesign included
- ▼
- Custom shade consultation and photos (if requested)
- ▼
- Approved wax up design models, prep guides, temp matrix sent back to dentist

Dr. Appt 2

-  Prep teeth, verifying clearances with prep guides
- ▼
-  Document stump shades with shade guide and photos
- ▼
-  Use temporary matrix to fabricate temporaries
- ▼
-  Adjust and refine temporaries to patients satisfaction.
- ▼
-  Scan patient approved temporaries, if not sure give them a few days to wear and adjust until approved
- ▼
-  Finalize shade, incisal translucency, anatomy, surface texture, material, layered/monolithic.
- ▼
-  Fully complete Dentek Cosmetic Lab Slip with all details

► **Recommend PATIENT SIGNOFF** at this step to avoid costly remake charges

Lab Step 2

- Once ALL required information is received, final design on prep scan completed and returned for preview through portal
- ▼
- One redesign included
- ▼
- Only when final design is approved by the doctor, in lab turnaround time begins - 10 days layered, 8 days monolithic
- ▼
- Any changes made after approval charged 50% after milling, 100% after ceramics

Dr. Appt 3

-  Seat restorations, adjust
- ▼
-  **Recommend patient signoff BEFORE cementation!**
If unsure temp cement and adjust until approved
- ▼
-  Once approved, permanent cementation and polish
- ▼
-  Any changes or remakes after cementation charged at 100%
- ▼
-  Scan for nightguard if requested
- ▼
-  schedule appt with lab 2 weeks later for FREE custom 3D printed nightguard and after photos

Lab Step 3

- Deliver FREE nightguard to patient and take after photos
- ▼
- Before and after photos provided to dentist for website / social media. Happy patients asked to give five-star review with photos for doctor and lab
- ▼
- Dentek will post before/after photos and shout out your practice with permission from doctor and patient